A complete scar care concept

silicone - drukkledij - hydratatie
silicone - vêtement compressif - hydratation
silicone - compression - hydration
Silikon - Druckkleidung - Hydratation
in silicone - indumenti di compressione - idratazione

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na - après
after - nach - dopo

uw mening / resultaat
votre opinion / résultat
your opinion / result
Ihr(e) Meinung / Resultat
il vostro parere / risultato
@
www.caroskin.eu

NL

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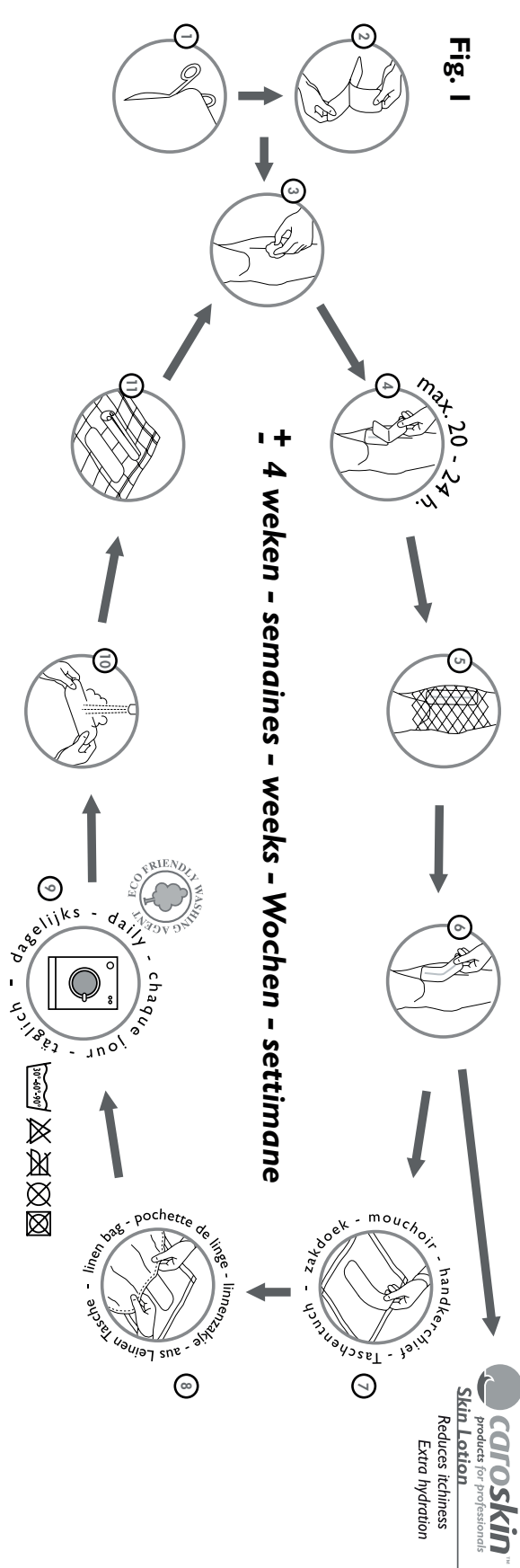
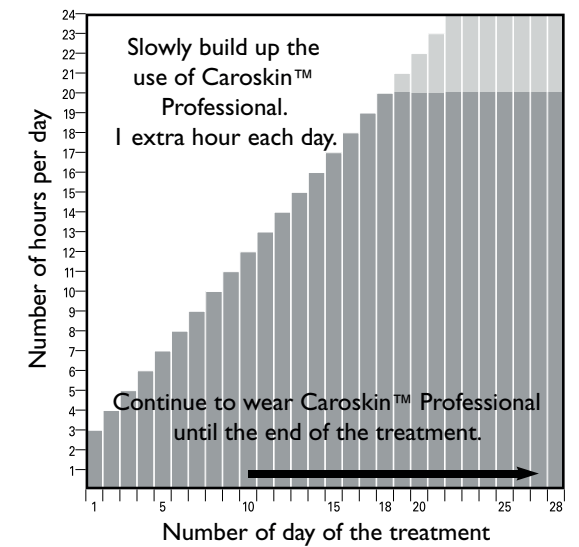
Fig. 2

Indications and action

- The *Caroskin™ Professional* Silicone Sheets are developed for the treatment and/or prevention of new and existing keloid and hypertrophic scars.
- The *Caroskin™ Professional* Silicone Sheets improve the elasticity of existing scars and reduce redness and raised scarring. The *Caroskin™ Professional* Silicone Sheets are also to be used as a buffer that prevents direct friction.

Description of the Caroskin™ Professional gamma

The *Caroskin™ Professional* silicone sheets are

- Self-adhesive silicone bandages, to be cut to size, for scar treatment; available in a *Light* (thin and elastic) version and in an *Elastic* (thick and elastic) version.
- The *Caroskin™ Professional* silicone sheets are made of 100% medical silicone gel, a **DOUBLE** elastic layer and a protective layer.

A range of *Caroskin™ Professional* Handproducts and special silicone bandages, to be cut to size, are used for post burn treatment, post (plastic) surgery and dermatology applications. The silicone gloves are made of 100% medical silicone gel and an elastic polyamide material.

Caroskin™ Cleaning Soap is an especially developed cleansing agent for the *Caroskin™ Professional* products.

Caroskin™ Skin Lotion, a comfort enhancing emulsion for the skin, with caring properties that does not interfere with the characteristics of the products.

Length of use

The *Caroskin™ Professional* Silicone Sheets for 2 to 8 weeks, depending on the site requiring treatment and conditions. The *Light* products lasts in general from 2-4 weeks, the *Elastic* version last in general for up to 8 weeks.

Directions for using and practical tips for the Caroskin™ Professional Silicone Sheets

- Cut the *Caroskin™ Professional* Silicone Sheets with rounded edges and cover the scar completely, with an overlap of approximately 2 cm. (fig. I-1)
- Remove the protective film from the sheet and stick the adhesive side onto the scar. (fig. I-2)
- Increase the use of *Caroskin™ Professional* Silicone Sheets with 1 hour per day up to a total of 20 hours. (fig.2)
- You can use tape or a bandage to make sure that the *Caroskin™ Professional* Silicone Sheets stay in place. (fig. I-5)
- After slowly removing the *Caroskin™ Professional* Silicone Sheets, it is best to rub *Caroskin™ Skin Lotion* into the skin for intense hydration.
- When the adhesiveness is insufficient, clean the *Caroskin™ Professional* silicone sheet according to the care advice. Washing the product will increase its adhesiveness!

Wash & Care Advice for the Caroskin™ Professional Silicone Sheets

- Rub a few drops of an ecological friendly washing agent or *Caroskin™ Professional* into the *Caroskin™ Silicone Sheets*.
- The sheets lose their adhesive power after contact with the cleaning agent and/or water. Or wash the silicone sheet in the washing machine. (fig I-7-8-9)
- Rub well to remove all dirt and rinse thoroughly under warm, running water.
- Place the (curled) *Caroskin™ Professional* Silicone Sheets on a dry tea-towel, sticky side up, to dry and let it uncurl. When dry, the adhesiveness will return. The better you wash and clean, the better the adhesiveness will be!
- Only use *Caroskin™ Professional* Silicone Sheets when they are fully dry.

Directions for using and practical tips for the Caroskin™ Professional Handproducts (silicone glove and fingertap)

- Respect the build up period/schema
- The silicone glove/fingercap is worn if there is scar formation on the hand or on the fingers. The action of the silicone glove makes the skin more supple and promotes the mobility of the joints.
- To pull the glove on, first fold back the edge of the glove.
- Put some *Caroskin™ Skin Lotion* on the hand and in the glove and pull it on. *Caroskin™ Skin Lotion* also promotes the durability of the silicone layer.
- In any case, **ALWAYS** use the *Caroskin™ Skin Lotion* for pulling on the silicone glove.
- After pulling the glove on, turn the edge of the glove back into place.

Wash & Care Advice for Caroskin™ Professional Handproducts (silicone glove and fingertap)

- Turn the glove/fingercap inside out, rinse and rub in with an ecological washing agent or a few drops of the *Caroskin™ Cleaning Soap*. Or wash the silicone glove in the washing machine at 40°C.
- Rinse thoroughly with mild warm running water and let the glove/fingercap dry inside out.
- **ONLY** and **ALWAYS** use the *Caroskin™ Skin Lotion* for pulling on the glove or fingertaps.

Precautions:

- **NEVER** use the *Caroskin™ Professional* Silicone Sheet on an open wound!
- *Caroskin™ Skin Lotion* and *Caroskin™ Cleaning Soap* are specially developed for further enhancing the comfort of the treatment.
- **NEVER** use other salves, creams, gels or lotions in combination with the *Caroskin™ Professional* Silicone Sheet, unless this is explicitly prescribed by your doctor.
- Always consult your doctor concerning skin irritation or skin problems. In many cases a more thorough hygiene of the product and a prolonged build up period of the use of the products will be helpful.
- *Caroskin™ Professional* products are for external use only.
- *Caroskin™ Professional* products are produced to exacting quality standards. If the *Caroskin™ Professional* silicone sheet should still lose silicone through wear-and-tear at the edges, you can trim the edges by sticking the silicone bandage on a piece of plastic foil and then trimming the edges.
- Never dry the *Caroskin™ Professional* silicone sheet in the sun or near a source of heat.
- *Caroskin™ Professional* products are entirely safe for treating children's scars. Nevertheless, keep *Caroskin™ Professional* products out of the reach of children.
- *Caroskin™ Professional* products are not sterile.

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